IDRACARE®

Gel hidratante vaginal

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What is IDRACARE® vaginal moisturizing gel?

EN

IDRACARE® is a non-hormonal vaginal gel with moisturizing properties for the vaginal mucosa. Its use is recommended when there is a decrease in natural vaginal fluid, in circumstances such as:

- Hormonal changes during the menstrual cycle, pregnancy, puerperium, lactation and peri and post menopause
- Medical treatments, such as the use of oral or vaginal contraceptives, radiotherapy
- Intense exercising, smoking and stress
- Alterations of the vaginal microbiota

Vaginal dryness produces annoying symptoms such as itching, irritation, stinging or dyspareunia (pain during sexual intercourse). With the use of **IDRACARE®**, the discomfort caused by the vaginal dryness decreases.

Its formula incorporates several components that act at the level of the vaginal mucosa, recovering the epithelium's adequate level of hydration and maintaining its normal characteristics of flexibility.

Dosage: Its use is recommended twice a week, for 3 months. If the symptoms do not improve, it is recommended to consult your doctor.

How to use: It is recommended to use the gel at night before going to bed.

Idracare® is supplied in single-use vaginal cannulas. Any cannula that appears open or broken should be discarded. The cannula is designed to aid in the product's application. The cannula should be inserted into the vagina, then press to release its contents. Then, remove it from the vagina, while continuing to press the cannula, in order to not suction out the product already applied.

It is possible that a small amount of gel remains in the cannula. This does not mean that a dose less than that recommended has been administered, as the cannula releases a sufficient amount to cover the cervical-vaginal mucosa (3 ml). Once the cannula is used, it should be discarded. A cannula should not be reused nor shared between two patients.

Presentation:

- Package with 8 single-dose cannulas.
- Package with 16 single-dose cannulas.
- Package with 1 single-dose cannula (sample format, non-commercial).

Each cannula contains 5 ml of gel.

IDRACARE® contains the following components:

- **Hyaluronic acid niosomes:** Its main effect is the superficial moisturizing of the tissues, providing elasticity to the skin and mucosa.

- **BioEcolia®:** Prebiotic (Alpha-oligoglucan) that stimulates the growth of the beneficial bacterial flora and rebalances the microbiota in the vaginal zone.

- **Aloe vera:** Moisturizing, re-epithelizing and healing effect on the vaginal mucosa.

Warnings:

Do not use in people with known hypersensitivity to any of its ingredients. If you feel discomfort after application, stop using it and seek medical advice.

Improves sexual intercourses. Compatible with condoms.

- Fragrance-free • Parabens-free • pH=5

Store the product in cool, dry place.
Keep out of reach of children.

Distributed in UK:
PROCARE WOMEN HEALTH LTD.
271 High Street
Berkhamsted Herts HP411A - England